

Accutome A-Scan Serial Link Instructions Downloading Text Only to PC

Within the Set Up Screen on the Accutome A-Scan

*change "printer option" to NONE

*change "serial link" to GENERIC TEXT ONLY

*change "serial baud" to 115200 8-N-1

1. Connect the cable (serial cable) between Accutome A-Scan and Com 1 of PC
2. On PC go to start->programs->accessories->communication->hyper terminal
3. In Hyper terminal folder, double click Hyper terminal.exe
4. Enter a name for the file and click "ok"
5. Set "Connect using" to "Direct to COM1" and click "Ok"
6. Select the following settings:
Bits per second=115200
Data bits=8
Parity=none
Stop bits=1
Flow control=none
7. Click "ok"
8. Now you can download by pressing the print functions on the Accutome A-Scan from the measure screen
9. To print repeat steps 2 and 3, click "cancel", go to "File", from the toolbar select "Open", double click on the file desired to open it, then go back to the toolbar, select print and click "Ok"