

Accutome Connect™ Conversion Instructions

REV -




Converting Files for Connect Software


Quick Guide

This guide shows how to convert pre-existing Accutome A-Scan Plus Connect, B-Scan Plus, and UBM Plus patient files to be viewed in the Connect Software.

- Step One:** Locate and double click on **Accutome Conversion Tool** icon.

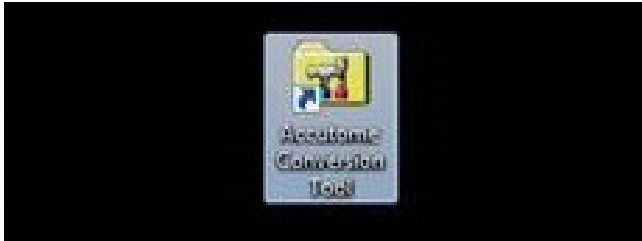


- Step Two:** Select Old File/Folder location for Exams you would like to convert.

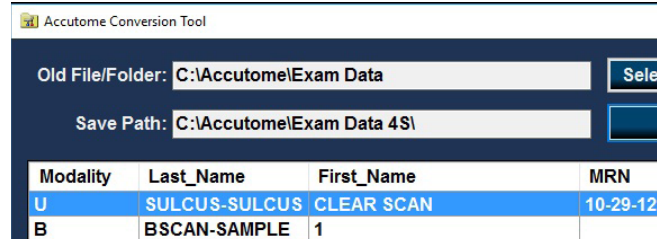

[Selecting the **A-Scan** button will show the default A-Scan folder on the computer]

[Selecting the **B-Scan/UBM** Button will show the default B-Scan/UBM folder on the computer]

[If the patient folder is in a separate location, use the **Select** button to search for folder]

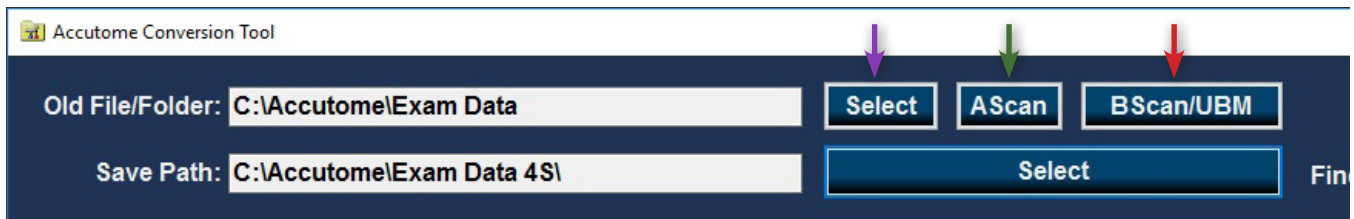

- Step Three**

A-Scan



- After Selecting the **A-Scan folder** in **Old File/Folder** section. Select all exams you would like to convert for Connect Software.

[To select multiple files, press and hold the **Ctrl** key while making selections with the mouse]

Note: A-Scan and B-Scan/UBM exams need to be converted separately.


B-Scan/UBM

- After Selecting the **B-Scan folder** in **Old File/Folder** section. Select all B-Scan/UBM exams you would like to convert for Connect Software.

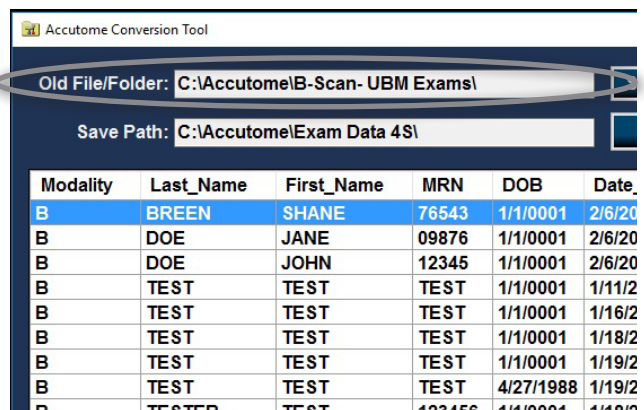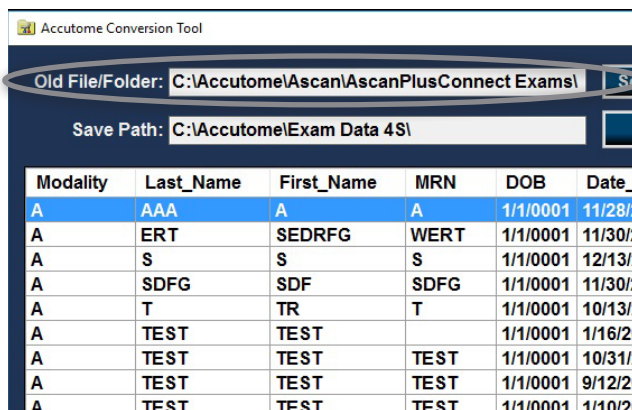


Installation Instructions


- Step Four:** Once all desired exams are selected, Click **Convert** icon.

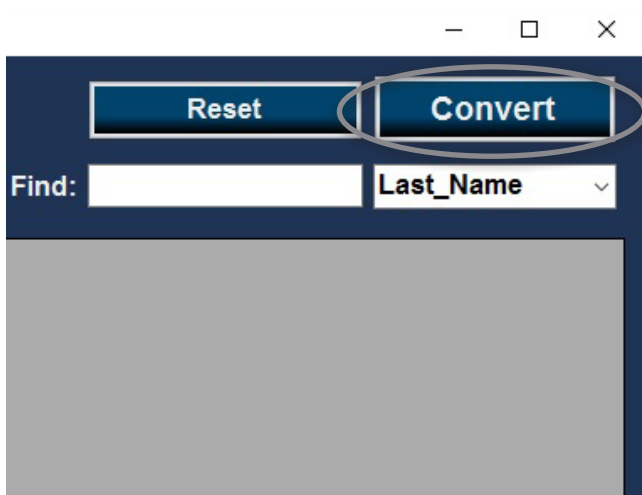


- Step Five:** Conversion will begin (Allow time for all files to convert) once copied, Click **Ok**.

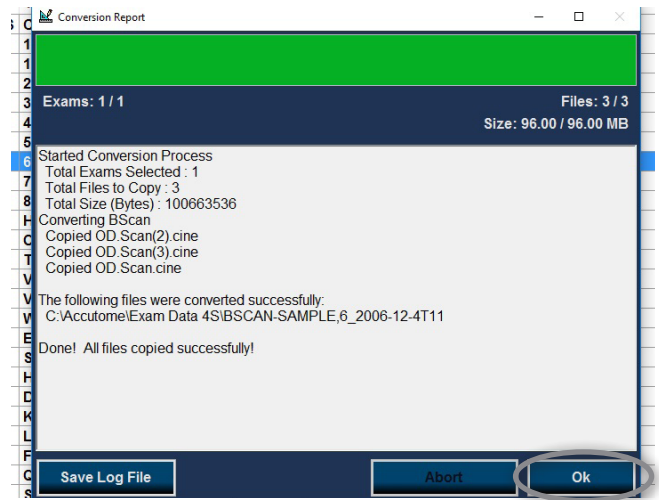

- Step Six:** Close Conversion Tool.
- Step Seven:** Launch Connect Software Icon

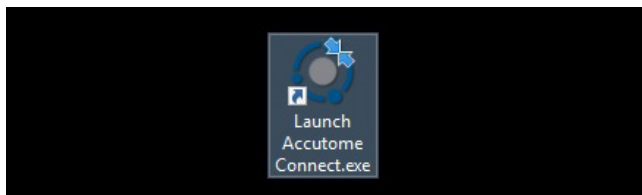

- Step Eight:** Select **Existing Patient** icon
- Step Nine:** Converted Patients will be available to Select and Review in Connect Software.

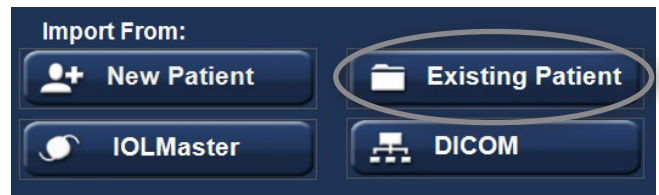

For assistance with the device upgrade or any other questions, please feel free to contact Accutome Diagnostic Support at diagnostics@accutome.com or 610.889.0200



1-800-979-2020 • 610-889-0200 • FAX 610-889-3233 • info@accutome.com • www.accutome.com

Total Solutions Provider for all of your ophthalmic needs.

ACCUTOME SURGICAL

Diamond Knives, Steel Knives, Sutures and Handheld Surgical Instruments.

ACCUTOME CLINICAL

Office Medications and Supplies.

ACCUTOME DIAGNOSTIC

A-Scans, B-Scans, Pachymeters, Tonometer and UBM.

ACCUTOME REPAIR CENTER

- Quick turnaround for most brands of Diamond Knives and Handheld Instruments.
- Diagnostic Equipment.

ORDERING

Call toll free (800) 979.2020 US and Canada
(610) 889.0200 International
+31 (0) 485.350300 Europe
Monday thru Friday 8:30AM - 8PM EST
Fax: (610) 889.3233
Email: info@accutome.com
Web: www.accutome.com

SHIPPING

Ground - Via UPS Ground Service
Air - Via best method

Saturday - Orders must be placed before 3PM EST
Friday to ensure Saturday delivery.

Payment should be made by check, Bank Transfer, Visa, MasterCard or American Express.

CONTACT US



WATCH RESOURCES

youtube.com/c/accutome



LIKE US

facebook.com/pages/Accutome-Inc



TWEET US

twitter.com/accutome



FOLLOW US

[linkedin.com/company/accutome-inc.](http://linkedin.com/company/accutome-inc)



GOOGLE+

plus.google.com/+Accutome/



Distributed by:

