

## Upgrading **B-Scan Plus/** **UBM Plus** On Windows 10

REV - A



**ATTENTION:**

PLEASE READ ALL ENCLOSED INFORMATION PRIOR TO USING ACCUTOME DIAGNOSTIC EQUIPMENT.

IF A LAPTOP HAS BEEN PURCHASED FROM ACCUTOME, ALL SOFTWARE NECESSARY TO RUN THE ACCUTOME SYSTEM IS PRE-INSTALLED ON ACCOMPANYING LAPTOP.

ACCOMPANYING SOFTWARE DISK IS TO BE USED IF INSTALLING SOFTWARE ON NON-ACCUTOME SUPPLIED COMPUTER.

**INDEX**


<b>Update B-Scan/UBM Software.....</b>	<b>3</b>
If Edge is Your Default Browser .....	3
If Internet Explorer is Your Default Browser .....	3
Uninstall Previous Software .....	3

## UPDATE B-SCAN PLUS/UBM SOFTWARE


To navigate to B-Scan Plus/UBM Software Version 4.07.02 for 64-bit download page, click the link provided in update email or Click Here for B-Scan and Here for UBM.

### If Edge is Your Default Browser:

1. **Step One:** Click [Download](#).

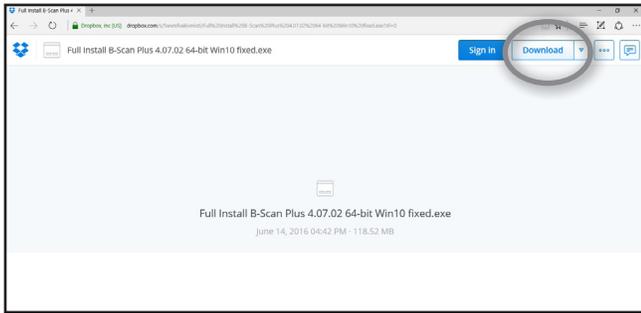


2. **Step Two:** Download will begin. When download is finished, exit the [Browser](#).

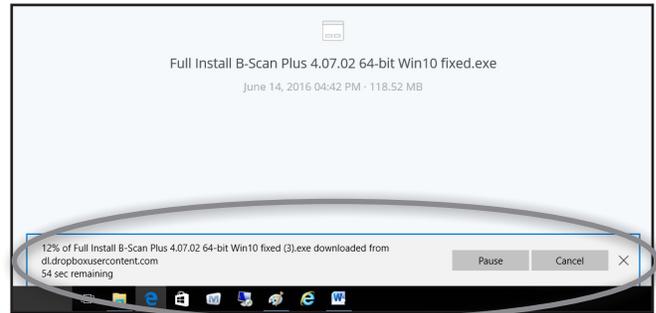


### If Internet Explorer is Your Default Browser:

1. **Step One:** Click [Download](#).

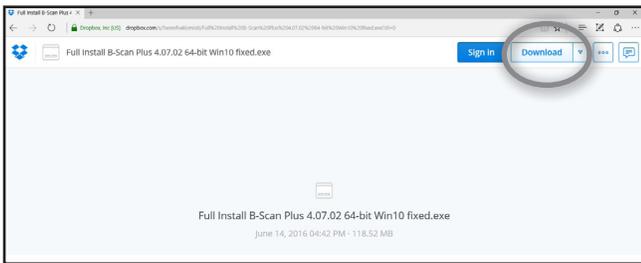

2. **Step Two:** When asked to [Run](#) or [Save](#), click [SAVE](#)

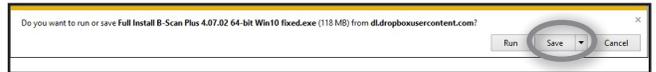

3. **Step Three:** Download will begin. When Download is finished, exit the [Browser](#).




### Uninstall Previous Software:

While the upgrade is downloading, uninstall the previous software.


**NOTE:** The Software is designed to be uninstalled and upgraded without losing any patient data. As a

precaution, it is recommended that any patient data be backed-up prior to uninstalling the software. **\*\*If you have any questions about this step Please Contact Accutome\*\***

4. **Step Four:** Move cursor over the bottom left hand corner of the screen over the [Windows Icon](#).

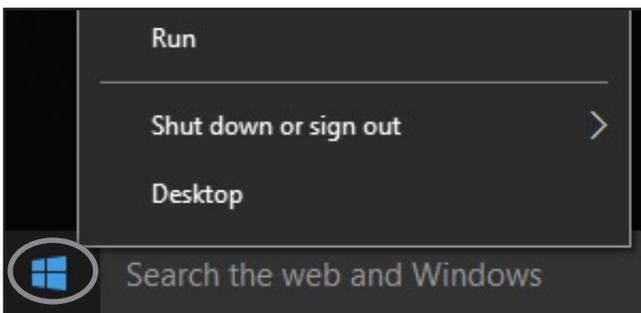

5. **Step Five:** Right click on the [Windows Icon](#). Select [Programs and Features](#).

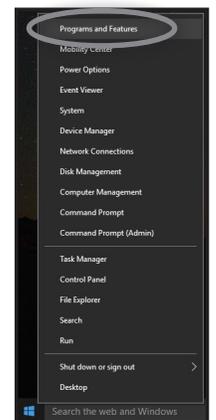


6. **Step Six:** In Programs and Features, select B-Scan Plus Software or UBM Plus software. Click Uninstall.




7. **Step Seven:** After successfully uninstalling the software, close all open windows and navigate back to the Desktop.


8. **Step Eight:** Locate the File Explorer Icon on the Desktop and open.

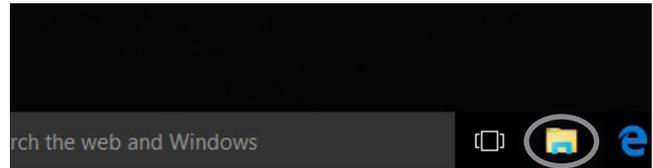


9. **Step Nine:** In File Explorer, select the Downloads Folder.

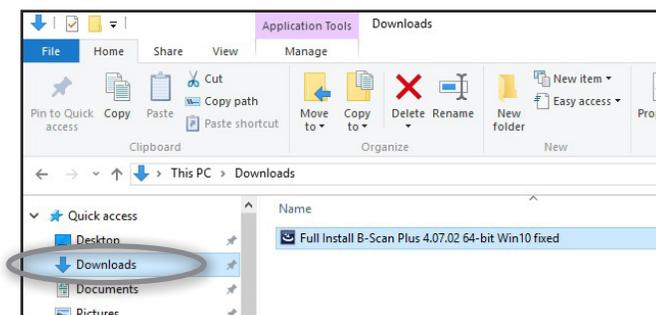


10. **Step Ten:** In Downloads, right click on the file [Full Install B-Scan or Full Install UBM] and select the option to Run as administrator,

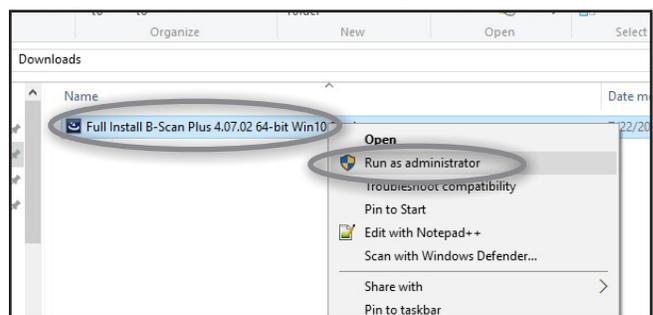

11. **Step Eleven:** Click More Info.

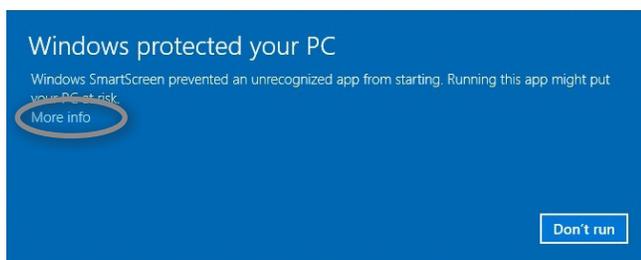

12. **Step Twelve:** Click Run Anyway.

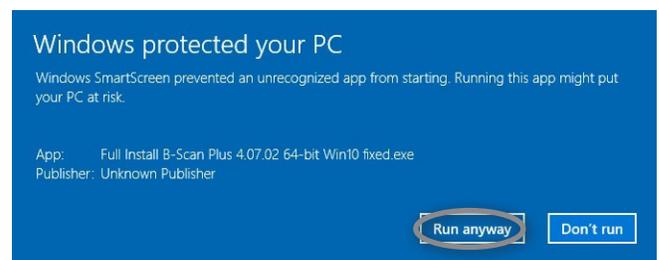

13. **Step Thirteen:** Follow the installation prompts until installation is complete.



14. **Step Fourteen:** Plug in the B-Scan or UBM and allow the computer to recognize the Driver.

15. **Step Fifteen:** Launch the Program and click **New Exam** to begin a new patient.

For assistance with the device upgrade or any other questions, please feel free to contact Accutome Diagnostic Support at [info@accutome.com](mailto:info@accutome.com) or 610.889.0200





1-800-979-2020 • 610-889-0200 • FAX 610-889-3233 • info@accutome.com • [www.accutome.com](http://www.accutome.com)

## Total Solutions Provider for all of your ophthalmic needs.

### ACCUTOME SURGICAL

Diamond Knives, Steel Knives, Sutures and Handheld Surgical Instruments.

### ACCUTOME CLINICAL

Office Medications and Supplies.

### ACCUTOME DIAGNOSTIC

A-Scans, B-Scans, Pachymeters, Tonometer and UBM.

### ACCUTOME REPAIR CENTER

- Quick turnaround for most brands of Diamond Knives and Handheld Instruments.
- Diagnostic Equipment.

### ORDERING

Call toll free (800) 979.2020 US and Canada  
(610) 889.0200 International  
+31 (0) 485.350300 Europe  
Monday thru Friday 8:30AM - 8PM EST  
Fax: (610) 889.3233  
Email: info@accutome.com  
Web: www.accutome.com

### SHIPPING

Ground - Via UPS Ground Service  
Air - Via best method

Saturday - Orders must be placed before 3PM EST  
Friday to ensure Saturday delivery.

Payment should be made by check, Bank Transfer, Visa, MasterCard or American Express.

## CONTACT US



### WATCH RESOURCES

[youtube.com/c/accutome](http://youtube.com/c/accutome)



### LIKE US

[facebook.com/pages/Accutome-Inc](http://facebook.com/pages/Accutome-Inc)



### TWEET US

[twitter.com/accutome](http://twitter.com/accutome)



### FOLLOW US

[linkedin.com/company/accutome-inc.](http://linkedin.com/company/accutome-inc)



### GOOGLE+

[plus.google.com/+Accutome/](http://plus.google.com/+Accutome/)



Distributed by: