
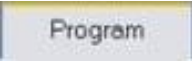








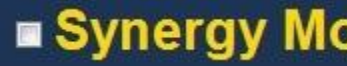
A-Scan Connect Synergy

Setup Guide

1. Power on IOL Master
2. Click on the **Settings** Icon in the bottom right hand corner (Wrench Icon) 
3. Select the **Programs** tab (Second tab in the settings screen) 
4. Insert Accutome **USB Disk** into the IOL Master 
5. In the **Programs** tab on the IOL Master select **Update** 
6. Follow the Install prompts until installation is complete.
7. Restart the IOL Master.
8. Select the **i** icon in the bottom right hand corner of the IOL Master screen 
9. On the next screen (About IOL Master) find the name of the IOL Master labeled

Computer Name (IOLM#####)

Computer name: IOLM1039222

10. Remove USB Disk and restart the IOL Master..
11. Connect the Laptop/PC and the IOL Master either by network or switchbox/router
12. Connect **A-Scan Plus Connect** unit to computer via USB port 
13. Launch **A-Scan Plus Connect** software on your PC or Laptop
14. Select the **Setup** tab on the top right hand side of the A-Scan software 
15. Select the **Import/Export** tab under the **Setup** tab 
16. Check the box next to **Synergy Mode** 
17. Double check the LAN IP Mode is set to Automatic

Local Area Network IP Mode **Automatic**

18. Enter the **Computer Name** you found on your IOL Master in the box labeled **Name of IOL Master** (IOLM#####)

Name of IOLMaster: (Ex: IOLMxxxxxxx)

19. Click the **Connect to IOLMaster** button.

Connect to IOLMaster