

Accutome Connect™

DICOM Quick Reference Guide

www.accutome.com

3222 Phoenixville Pike, Bldg 50 | Malvern, PA 19355 USA | info@accutome.com 800.979.2020 | 610.889.0200 | F: 610.889.3233 | European Office +31 (0) 485-350300



DICOM Worklist and PACS

- 1. Launch Accutome Connect software.
- 2. On the Connect Home Screen select the DICOM Button.
- 3. "Search Worklist" will search Today's Date (uncheck box to search thru all dates)
- 4. Click "Search Worklist".
- 5. Patients will load in the list, Double Click on desired patient.

6. Patient's information will load on the home screen.

- 7. Within the "Indication of Scan" window, scans requested for the patient will be listed.
- 8. Proceed to Modality that is requested in "Indication of Scan" window by choosing the following icons:

Search Worklist



Accutome Connect Version 8.00.00

UDI: 00897826002284

Status

Ô

Exit

💻 DICOM

Network Settings

CCUTOME

Indication of Scan:


B-Scan

- A-Scan
 - Capture scans
 - Proceed to IOL calculation Screen
 - Enter Information and perform Calculations
 - Save Changes
 - Export to DICOM

L'u	an :	Alcon monofocale			·	Manage	Physici	n Lenses					Name					
-	-A	SN60WF			SA60AT 118.40 5.20 1.450 521			MA60AC 118.40 5.20 1.450 5.21			SN6AT4		20					
	A-Const	118.70 5.37 1.620 5.40									119.00 5.55						-	
	Hoffer Q													K Index: 1.3315			OD/C	S)
	Holladay										1.790							
	SRK/T Haigis	1714	0.400	0.100	1.527	0.400	0,100	1.527	0.400	0.100	1,902	0.400			ype: Phakic			
	SRK II	1./14	118.70		1.527	118.4			118.40		1.902	119.0		AXL	26.88	m		
	Binkhorst II		5.37	,	5.20			5.20			5.55			42.46		Rx Surg No -		
s	Billichorsch	Power Refr		Power Refr		Power Refr		Power Refr			43.80							
0																		
		12.5		-1.48	12.0		-1.28	12.00		-1.28	12.5		-1.34	Target	-2.00		xport to Di	com
		13.0	-	-1.81	12.5	-	-1.61	12.50		-1.61	13.0	-	-1.67	He	offer Q			
		13.5		-2.15	13.0		-1.95 -2.30	13.00		-1.95	13.5		-2.00	Но	iladay	Sci	an AXL	
		14.0		-2.49	13.5		-2.50	14.00		-2.50	14.0		-2.55	S	RK/T	1	26.92	
		14,5	0	-2.85	14.0	U	-2.04	14.00		-2.04	14.5	U	-2.07	н	aigis	2	26.89	
														-	RKI	4	26.85	
	Target	13.28			13.07			13.07			13.50		Bink	chorst II	5	26.87		
	Emme	10.21			10.04			10.04			10.39	10.39			mpare	Av		
																		- 1.
														0	B -			O

- B-Scan/UBM
 - Scan patient- Review scan and take snapshots of the scan
 - Create Report by selecting snapshots desired in the report and hit preview
 - Exit Preview and Select Export report to DICOM
 - To only send images to DICOM without report, Highlight all desired Images and select "export selected to DICOM"



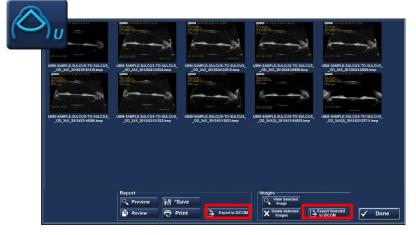

For assistance with the device upgrade or any other questions please feel free to contact Accutome diagnostic support at <u>diagnostics@accutome.com</u> or 610-889-0200.