

Symbols

Symbolos • Simboli • Symbole • Simbolos



Catalog Number

Reference • Numero di catalogo • Bestellnummer • Numero de catalogo



Lot Number

Numero do lot • Numero di lotto • Ch-B. • Numero de lote



See Insert

Voir notice a l'interieur • Vedere insert • Siehe Beilage • Veá la hoja intercalada



Do Not Reuse

Ne pas reutiliser • Non riutilizzare • Nicht wiederverwende • No reusar



Use By Date

Utiliser avant le (date) • Scadenza • Verfallsdatum • Use antes de la fecha



Keeler

3222 Phoenixville Pike

Building 50

Malvern, PA 19355 USA

Toll Free: 1-800-523-5620

Tel: 610-353-4350

Fax: 610-353-7814

Email: customerservice@keelerusa.com

Website: www.keelerusa.com

Keeler Ltd.

Clewer Hill Road, Windsor

Berkshire, SL4 4AA, UK

Tel: +44 (0) 1753 857177


Fax: +44 (0) 1753 827145

Email: info@keeler.co.uk

Website: www.keeler.co.uk

Keeler

— A world without vision loss —

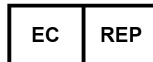

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



AccuTip

Covers

AccuPen® Tip Covers

Product Information and Instructions



Couvre-Pointe AccuTips

Informations et instructions concernant le produit



Copertura per la Punta AccuTips

Informazioni sul prodotto ed istruzioni per l'uso



AccuTips Spitzenhülle

Produktinformation und Gebrauchsanweisung



La Cubierta de Punta AccuTips

Instrucciones e informacion sobre el producto

Keeler

— A world without vision loss —



A Halma company

AX9919R REV C

Instructions for use of AccuTip covers with the AccuPen

The AccuTip cover is applied over the tip of a handheld tonometer prior to performing tonometry. The tip cover is used for protection of the patient's eye and instrument tip.

Preparation of the instrument:

Before measuring intraocular pressure, cover the tip of the AccuPen with an AccuTip cover.



Place cardboard sleeve over the tip of the AccuPen and slide the cover down until it sits in the groove.



Make sure that the rubber cover is flat across the tip and not too taut or too loose. This could cause issues with readings.



Tip Cover Fits Correctly



Tip Cover too loose



Tip Cover too tight

Also compatible with the Reichert® Tono-Pen®

Notes:

- The AccuTip Cover can be used with most handheld applanation tonometers, such as the Keeler AccuPen and Reichert TonoPen.
- Corneal topical anesthetic is required with tonometry.
- Refer to the AccuPen instruction manual for further information.

Cautions/ Warnings:

- THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.
- An AccuTip cover must be applied to the handheld tonometer before performing tonometry. Failure to do so may result in abrasion of the cornea.
- It is imperative that a fresh AccuTip cover be used for each patient for the protection of the eye and the instrument tip, preventing the transfer of infection. AccuTip covers should be replaced after examination of each patient.
- Each box of AccuTip covers has a "USE BEFORE" date stamped on the box. We suggest use of our AccuTip covers before this date to guard against the possibility of shelf wear. The latex material used in the AccuTip covers can degrade. A degraded film may result in leakage of wetting solution and instrument damage. Examine each AccuTip cover for yellowing, cracks or a sticky texture prior to use. AccuTip covers should be stored between 35° and 80° Fahrenheit (2° - 27° Celsius).