

B-Scan Pro/Vet Recommended Computer Specs

1. **Processor:**
Intel[®] Core[™] 2 Duo 2.00 GHz or higher
2. **Memory:**
2.0 GB, DDR2 SDRAM
3. **Video:**
256 MB NVIDIA GeForce 8600M GT graphics card
WSXGA (1280 X 800 pixels or 1280 X 1024 pixels) monitor
15.4 inches or higher screen size.
4. **Hard Disk:**
Minimum 80 GB, 7200 RPM, recommended 160 GB or higher for high patient flow site (more than 30 B-Scans per month).
5. **Optical Drive:**
8X (or higher) DVD+/-RW with control software (needed for archiving)
6. **USB Ports**
Minimum of 3 USB 2.0
7. **Medical Grade Power Supply**
8. **Software**
64-Bit version is not currently supported
Microsoft Windows[®] XP Professional Edition, SP2
Microsoft Word[®] 2003 or 2007 (needed for report generation)
Microsoft Windows[®] Vista Business or Ultimate editions 32-Bit version.
9. **File System**
NTFS file System

***Does NOT support either Dual Monitors or Dual Video Cards**

B-Scan Pro/Vet Recommended Computer Specs

If the computer on which you intend to load the B-Scan Software does not meet or exceed the listed specifications, we highly recommend that you do not install the B-Scan Pro Software.

It is recommended that the computer used in conjunction with the B-Scan Pro system be dedicated for the sole use of Accutome's B-Scan. Installation of software not vital to the operation of the B-Scan is highly discouraged, as it may detrimentally affect the performance of the B-Scan Pro system. Additionally, accessing the Internet will greatly affect the performance of the B-Scan and open the system to viruses and other harmful external factors and is therefore not recommended by Accutome.