



4Sight™ Ophthalmic Ultrasound Diagnostic Unit System Protection

www.accutome.com

3222 Phoenixville Pike, Bldg 50 | Malvern, PA 19355 USA | info@accutome.com
800.979.2020 | 610.889.0200 | F: 610.889.3233 | European Office +31 (0) 485-350300



Turn Off System Protection 4Sight


1. Turn on 4Sight system
2. Click on the “Windows OS” icon in the bottom right hand corner of the screen.



3. Once at the desktop
 - Navigate to the Control Panel
 - Click on windows Start Icon
 - Select Control Panel



- Select “System and Security”

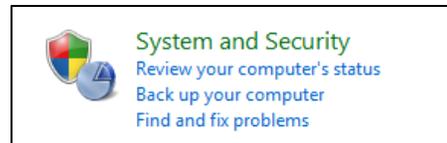


- Select “System”



- Select “System protection”




- In System Protection, Select “Configure”

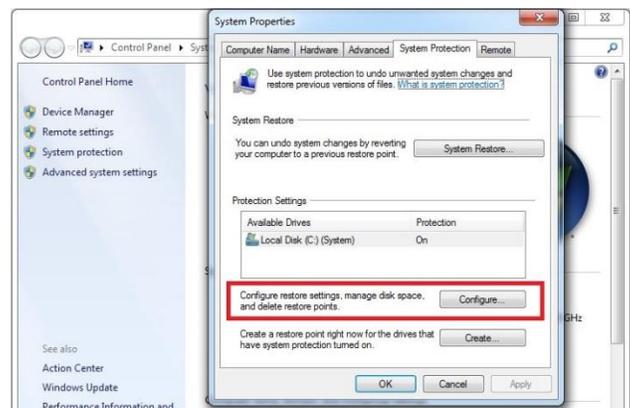


- Select Radio Button **“Turn Off Protection”**

- Select **“Apply”**

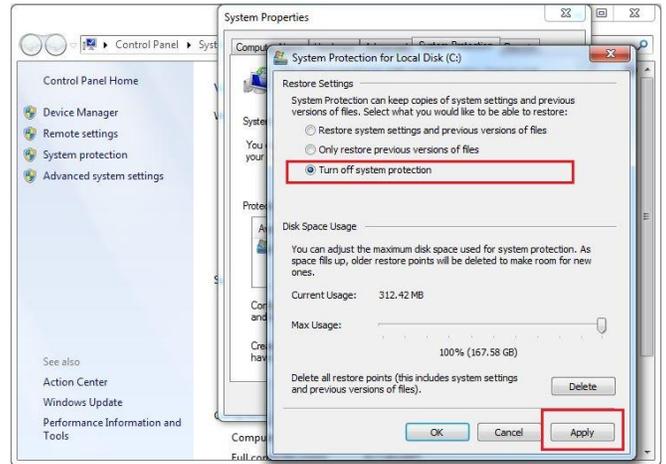

- Select **“Yes”**

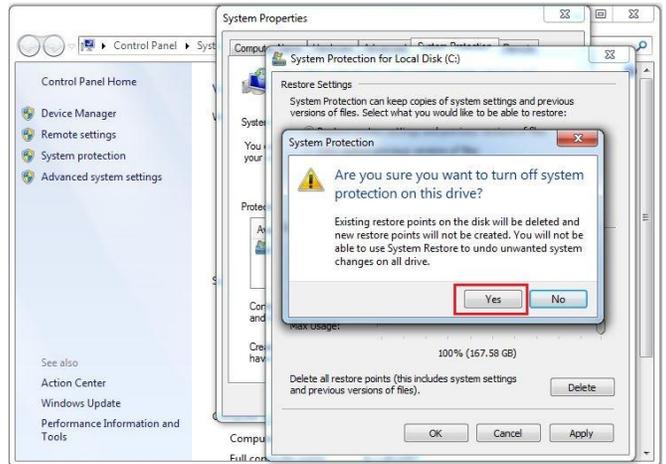

4. Re-boot the 4Sight system.

For assistance with the device upgrade or any other questions please feel free to contact Accutome diagnostic support at info@accutome.com or 610-889-0200.